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d

Ready to use disinfectant cleaner for rotary instruments

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- Ready-mixed solution
- Guaranteed protection against infection
- For clean instruments
- Odour neutral

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For the gentle cleaning and disinfection of steel drills, root canal and diamond instruments, milling tools and small-sized rotary instruments. Also qualified for the disinfections during ultrasonic cleaning. Effective against bacteria (including TbB/Mycobaterium terrae), fungi and viruses (HBV/HIV). DGHM-listed (German Society for Hygiene and Microbiology).

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## english

### Disinfectant effects:

- bactericide
- tuberculocide
- fungicide
- limited virucidal efficacy (HBV/HIV)

### Composition:

100g Bohricin® drill bath contains:

- 4.7g potassium hydroxide
- 7.9g ethanol
- 7.2g 2-propanol
- 0.4g formaldehyde
- 0.9g glyoxal
- excipients
- water

### Application instructions:

Insert used instruments in Bohricin®-bath immediately after use, in order that contamination not dry onto the surface. Do not mix with other substances, and after bath, rinse with water. The instruments may be used again after drying on pulp-towels. Caution: Do not touch instruments with fingers. Sealable tanks must not consist of polycarbonate. Bohricin® is also applicable for disinfection during ultrasonic cleaning.

### Required exposure time:

15 minutes  
Instruments placed in the bath for 2 days will suffer no damage. The bath should be renewed more frequently if instruments are heavily soiled.

### Special advice:

Prior to hot air sterilisation, instruments and burs should be thoroughly rinsed under running water after exposure to Bohricin® to avoid black discoloration. Plastic and aluminium components sensitive to alkalis and alcohols and drills with resin adhesives should not be placed in a Bohricin® bath.

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FAVODENT Karl Huber GmbH, Greschbachstr. 17, 76229 Karlsruhe