Instrumentendesinfektion

Instrument disinfectant

Desinfectantes para instrumentos

# Favosept®

Konzentrat

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PZN: 1 L: 1915227  
5 L: 1915233

**CE 0482****Efficacy spectrum:**

- bactericide
- tuberculocide
- fungicide
- limited virucidal efficacy (HBV/HIV)

**Composition:**

100g Favosept® - Concentrate contains:

9.9g bis (aminopropyl) dodecylamine  
8.4g didecylmethylpolyoxyethylammonium-  
propionate  
anionic tensides  
complex formers  
excipients  
water

**Application instructions:**

Insert instruments into working solution immediately after use. All surfaces and cavities requiring disinfection must be completely submerged in the working solution. Replace solution as fit to the contamination level.  
After disinfection, thoroughly rinse and clean instruments. For maintenance of hinges use instrument maintenance spray/silicon spray.

**The following Favosept® dilutions should be used:**

3% dilution  
Add water to 30 ml concentrate to make up a total quantity of 1 litre  
15 minutes reaction time

2% dilution  
Add water to 20 ml concentrate to make up a total quantity of 1 litre  
30 minutes reaction time

1% dilution  
Add water to 10 ml concentrate to make up a total quantity of 1 litre  
1 hour reaction time

Favosept® is also suitable for disinfection procedures in an ultrasonic bath.

**Special advices:**

The instrument disinfectant is acidly, therefore skin- and eye contact with dilution respectively concentrate is to avoid. Wash moistened body areas with plenty of water. Wear suitable gloves and eye protection.

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FAVODENT Karl Huber GmbH, Greschbachstr. 17, 76229 Karlsruhe