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Surface disinfectant

Desinfectantes de superficies

# FloraQUICK® Tücher

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Wipes for the cleaning and disinfection of medical devices (acc. to MDD) and small surfaces (acc. to BPD) in all hygiene sectors

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- Liquid is DGHM (German Society for Hygiene and Microbiology)-listed
- Extremely short exposure times
- Very tear-proof
- Phenol- and aldehyde-free
- Limited virucidal efficacy
- Pleasant scent
- Broad range of efficacy
- Quick drying

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Ready-to-use, alcoholic wipes for the rapid disinfection and cleaning of medical devices (acc. to MDD) such as straight and contra-angle handpieces and small surfaces (acc. to BPD) such as fixtures, walls and clothing. Effective against bacteria (incl. TbB and strains of MRSA), fungi and viruses (enveloped viruses such as HBV, HCV, HIV), the liquid is DGHM listed.

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Box containing 100 wipes,  
refillable Order No. 116

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**Efficacy spectrum:**

- bactericidal (incl. strains of MRSA)
- tuberculocidal
- fungicidal
- limited virucidal efficacy (inactivates enveloped viruses such as HBV, HCV, HIV)

**Composition:**

100g FloraQUICK solution contains:

24g ethanol  
37g 1-propanol  
excipients

**Application:**

Thoroughly wipe the areas to be disinfected using the disinfectant wipes and allow to act.

**Affective times:**

Inactivation of bacteria and fungi in accordance with DGHM<sup>1</sup>:

conc. - 30 sec. (at low contamination)  
conc. - 1 min. (at high contamination)

Inactivation of MRSA:

conc. - 30 sec. (at low contamination)  
conc. - 1 min. (at high contamination)

Inactivation of enveloped viruses:  
conc. - 30 sec.

**Special advices:**

Highly flammable!  
Keep away from sources of ignition - No smoking. Flashpoint in accordance with DIN 51755: 25°C.  
Keep out of the reach of children.  
Keep container tightly closed.  
Not suitable for use on materials, such as acrylic glass, that react sensitively towards alcohol.

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FAVODENT Karl Huber GmbH, Greschbachstr. 17, 76229 Karlsruhe