Reinigung & Pflege

Cleaning & Lubricating

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Steria®-Spray

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Cleaning and lubricating system for all high and low speed hand pieces and contra angles eng

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- Cleans, lubricates and protects in one simple step
- Used daily, it extends instrument service life

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Low viscosity, resin-free and non-acidic cleansing spray with lubricant additives.

Treatment with Steria Spray produces a protective film which reduces friction and wear.

Available with 3 interchangeable adapters for use on high and low speed handpieces and contra angles.

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250 ml spray-bottle Order No. 203
500 ml spray-bottle Order No. 204

Attachment 040 Order No. 040
Attachment 050 Order No. 050
Attachment 060 Order No. 060
Attachment 070 Order No. 070

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english

Composition:

Blend of non-resinous oils and solvents.

Application:

Treat instruments with Steria Spray prior to each sterilization run. For best results, repeat treatment 2-3 times per day.

Clean turbine or hand piece and contra angles every midday and evening by using spray for 1-2 seconds and remove any oil from the outside surface.

Clean turbines or hand pieces and contra angles by using spray for 1-2 seconds (till the oil leaks from the heading section), remove any oil from the outer surface and let it dry shortly afterward. Hold spray-bottle in a vertical position!

Special advices:

Pressurized container: protect from sunlight and do not expose to temperatures exceeding 50°C. Do not pierce or burn, even after use. Do not spray on a naked flame or any incandescent material. Keep away from sources of ignition - No smoking. Keep out of the reach of children.

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FAVODENT Karl Huber GmbH, Greschbachstr. 17, 76229 Karlsruhe