

Disinfectant for dental suction systems

- broad spectrum of activity
- high material compatibility for suction systems, amalgam separators and mouth rinse basins of any kind
- Aldehyde-free suction disinfectant
- Environmental compatible
- Efficient in utilisation
- Foam free
- Removes Biofilms very effectively

Extensive efficacy spectrum

- Bactericidal
- Levurocidal, (Candida albicans)
- Limited virucidal (against membrane covered viruses e.g. HIV, HBV, HCV)

Active ingredients:

100g of Favocid contain:

7,5g Dimethyldioctylammonium chloride

0,6g Benzyltrimethylammonium chloride

Exposure time:

Bactericidal (tested in accordance to VAH/DGHM), Fungicidal/Levurocidal

- organic load	1.0 %	60 min
	2.0 %	5 min
	3.0 %	2 min

Limited virucidal (against membrane covered viruses e.g. HIV, HBV, HCV)

- organic load	2.0 %	2 min
----------------	-------	-------

Unit	Article-No.
1l metered bottle	301
5l canister	302

On request we would be pleased to submit tailored conditions.

+49-721 62686-0
info@favodent.com

CE 0482 N-28103

Favocid Concentrate



Range of application:

Favocid is a high-quality quaternary bond-based disinfectant concentrate for dental suction equipment as well as mouth rinse basins and amalgam separators.

The suction system should be disinfected once or twice a day, preferable with 1 litre ready-to-use solution.



Suction Disinfection