

# Favosept® Concentrate



## Range of application:

Favosept® concentrate is a high-quality quaternary bond-based disinfectant for instruments with excellent cleaning power, especially for ultrasonic baths. This high-performance product is used for surgical and dental instruments of all kinds. Favosept® Concentrate has short exposure times - even against high concentration of germs - and provides comprehensive effectiveness.

## Expertises:

Prof. Dr. H. Schubert, Frankfurt a.M., DGHM-procedure, 12.2000

Prof. Dr. H.-P. Werner, Schwerin, DGHM-procedure, 07.2000

Prof. Dr. Gert Frösner, München, virucidal efficacy, 07.2000

## Disinfectant for medical instruments

- No alcohols or aldehydes
- Gentle on materials
- Very good cleaning properties
- Ideal for ultrasonic baths
- Non-corrosive

## Extensive efficacy spectrum:

- Bactericidal
- Tuberculocidal
- Mycobactericidal
- Levurocidal
- Limited virucidal (inactivates enveloped viruses such as HBV/HIV)

## Active ingredients:

100 g of Favosept Konzentrat contain:

9.9 % Bis-(3-aminopropyl)-dodecylamine

8.4 % didecylmethylpolyoxyethylammonium propionate, non-ion surfactant

## Exposure time:

Bacteria (incl. TbB/Mycobacterium terrae) and fungus (Candida albicans)

- organic load 3.0 % 15 min

2.0 % 30 min

1.0 % 60 min

HBV/HIV inactivation 3.0 % 15 min

- high organic load 1.0 % 60 min

Unit	PZN	Article-No.
1 L metered bottle	1915227	161
5 L canister	1915233	162

On request we would be pleased to submit tailored conditions.

+49-721-62686-0

info@favodent.com